ENGLISH
<p>BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.</p>

INTENDED USER PROFILE

The intended user should be the patient, a licensed medical professional, or the patient's caregiver.

The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS
<ul style="list-style-type: none">Support and compression following acute and chronic shoulder injury Provides proprioceptive and neuromuscular stimulation through compression (to support balance conditions and movements) Aids prevention of injury Post-operative immobilization (when used with the sling provided and under the direction of your health care professional) <p>Soft- good designed to provide mild protection and compression through elastic construction.</p>
CONTRAINDICATIONS
<ul style="list-style-type: none">Any known allergies to the medical device materials. Impaired sensation and movement disorders of the arms (e.g. diabetes mellitus) Impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support Vein thrombosis in the arm

WARNINGS AND PRECAUTIONS
<ul style="list-style-type: none">If you have skin conditions/trauma in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not apply on open wounds. Remove the support during longer periods of rest. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Handle with care: avoid fastening the strap's hook closure onto the knitted sleeve as it may cause damage to the appearance of the product when strap is removed. Do not use this device if it is damaged and/or packaging has been opened. <p>NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.</p>

APPLICATION INSTRUCTIONS
<ol style="list-style-type: none">Slide the sleeve section of the brace on your arm up to your shoulder. Adjust the sleeve until it is positioned correctly on the shoulder, the sleeve should extend an equal amount on the front and back of the shoulder. The pad should be resting on the bony protuberance at front and back of the shoulder. Guide one strap diagonally across the back of your body to the unaffected side and bring round to the front. Guide the other strap diagonally across the front of your body to the unaffected side and bring round to the back. Continue to guide the straps around your body back towards the affected side. Guide one strap through the eyelet on the end of other strap and fasten it loosely with the hook and loop closure. Tighten both straps until the required degree of stabilization is achieved. Fasten the tie straps securely. <p>NOTE: a sling is provided to support post-operative immobilization, keeping the elbow at 90 degrees.</p>

APPLICATION INSTRUCTIONS
<ol style="list-style-type: none">Slide the sleeve section of the brace on your arm up to your shoulder. Adjust the sleeve until it is positioned correctly on the shoulder, the sleeve should extend an equal amount on the front and back of the shoulder. The pad should be resting on the bony protuberance at front and back of the shoulder. Guide one strap diagonally across the back of your body to the unaffected side and bring round to the front. Guide the other strap diagonally across the front of your body to the unaffected side and bring round to the back. Continue to guide the straps around your body back towards the affected side. Guide one strap through the eyelet on the end of other strap and fasten it loosely with the hook and loop closure. Tighten both straps until the required degree of stabilization is achieved. Fasten the tie straps securely. <p>NOTE: a sling is provided to support post-operative immobilization, keeping the elbow at 90 degrees.</p>

CLEANING INSTRUCTIONS
<p>Machine washable at gentle cycle (max 40°C) with a mild detergent.</p> <p>Hook and loop should be closed for washing.</p> <p>Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.</p>

COMPOSITION
<p>Polyamide, Elastane, Polyester, Polyoxyethylène, Polyurethane, Cotton</p>

LATEX
<p>NOT MADE WITH NATURAL RUBBER LATEX</p>

WARRANTY D.J.Q. LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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